

Verifying that the patient is in the correct bed in OBIX

May 30, 2017


Important: Ensure the patient is in the correct OBIX bed **before** starting the fetal monitor to maintain the integrity of the tracing for quality assurance purposes.

Procedure:

1. Before starting EFM, select patient in Epic and open the chart.
2. From the Activity toolbar, click “**Start OBIX**” button to launch the system.
3. When OBIX launches, go to “**Patients**” menu and select “**In Unit**”.



4. The following screen should appear.

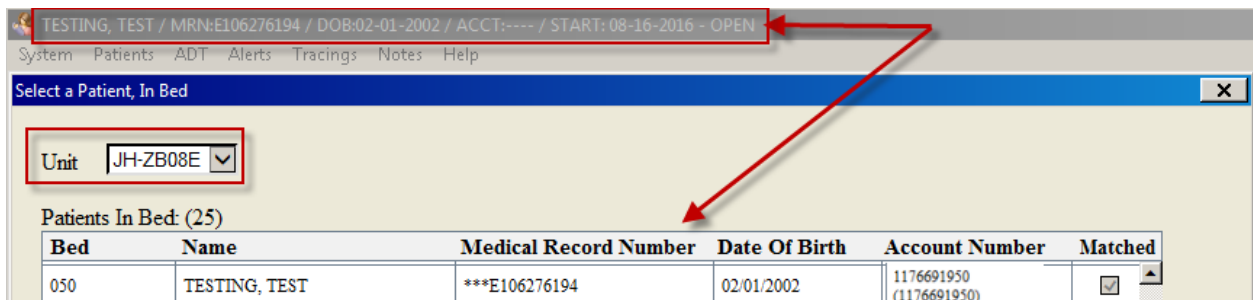

5. Find your patient bed on the list by scrolling and verify that **Bed, Name, MRN, DOB and Account Number** match Epic’s bed and location and demographics. Also, a check mark (✓) should be in the column titled “Matched.” It is **CRITICAL** that the OBIX and Epic bed assignments are identical.
6. After you verify that the patient is admitted to an OBIX bed, turn on the fetal monitor to begin capturing the fetal tracing.

